


Teva-Teriparatide Injection Pen

(teriparatide injection)

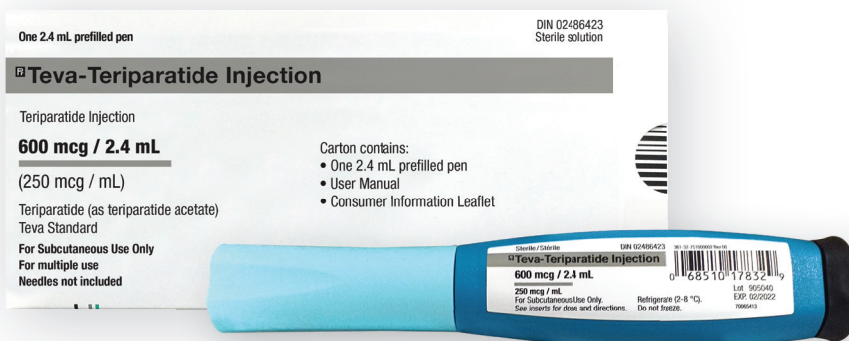



Before you use your new **Teva-Teriparatide Injection Pen**, please read the entire *User Manual* completely. Follow the directions carefully when using the **Teva-Teriparatide Injection Pen**.

Do not share your delivery device or needles as this may risk transmission of infectious agents.

Your Teva-Teriparatide Injection Pen contains 28 days of medicine.

Refrigerate the **Teva-Teriparatide Injection Pen** immediately after every use.

Teva-Teriparatide Injection Pen Parts*

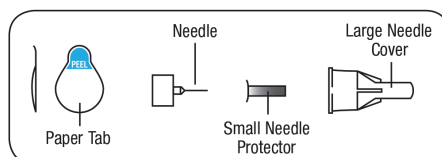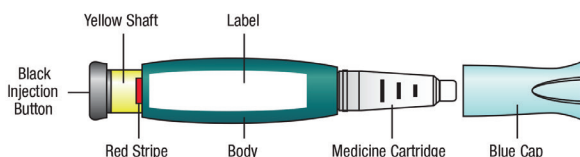

***Needles not included.** Becton, Dickinson and Company pen needles from 29 to 31 gauge are recommended for use with this device.

Wash your hands before every injection. Prepare the injection site as your healthcare provider instructed.

1 Pull off blue cap

After pulling off the blue cap, clean the rubber seal on the end of the TEVA-TERIPARATIDE INJECTION PEN cartridge with an alcohol swab.

2 Attach new needle




Pull off paper tab.

Push needle **straight** onto medicine cartridge.

Screw on needle clockwise until firmly attached.

Pull off large needle cover and **save it**.

3 Set dose

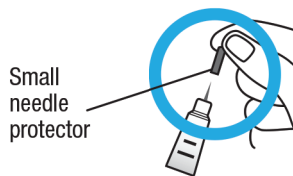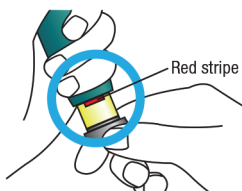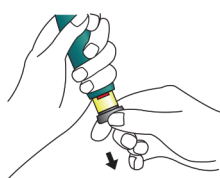

Pull out black injection button until it stops.

If you cannot pull out the black injection button see *Troubleshooting, Problem E*, on back page.

Check to make sure red stripe shows.

Pull off small needle protector and throw away.

4 Inject dose



Gently hold a fold of skin on your thigh or abdomen and insert needle straight into skin.

Push in black injection button until it stops. Hold it in and **count to 5 s-l-o-w-l-y**. You must wait until the count of 5 to make sure you receive the correct dose. Then pull the needle from skin.

5 Confirm dose

IMPORTANT




After completing the injection: Once the needle is removed from the skin, take your thumb off the black injection button. **Check** to make sure the black injection button is all the way in. If the yellow shaft does not show, you have finished the injection steps the right way.




You should **NOT** see any of the yellow shaft. If you do and have already injected the medicine, do not inject yourself a second time on the same day. Instead, you **MUST reset the TEVA-TERIPARATIDE INJECTION PEN** (see *Troubleshooting, Problem A*, on back page).


6 Remove needle

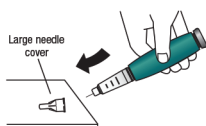


Put large needle cover on needle.

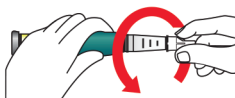

Unscrew the covered needle all the way by giving the large needle cover 3 to 5 counter-clockwise turns.

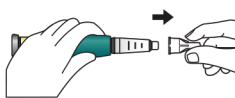


Pull off needle and throw away as directed by your healthcare professional.

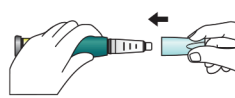

Push blue cap back on. Place TEVA-TERIPARATIDE INJECTION PEN in the refrigerator immediately after use.

Troubleshooting

Problem	Solution
<p>A. The yellow shaft is still showing after I push in the black injection button. How do I reset my TEVA-TERIPARATIDE INJECTION PEN?</p> 	<p>→ To reset the TEVA-TERIPARATIDE INJECTION PEN, follow the steps below.</p> <ol style="list-style-type: none"> If you have already injected, DO NOT inject yourself a second time on the same day. Remove the needle. Attach a new needle, pull off the large needle cover and save it. Pull out the black injection button until it stops. Check to make sure the red stripe shows. Pull off the small needle protector and throw away. Point the needle down into an empty container. Push in the black injection button until it stops. Hold it in and slowly count to five. You may see a small stream or drop of fluid. When you have finished, the black injection button should be all the way in. If you still see the yellow shaft showing, contact TEVA (see <i>Contact Information below</i>) or your healthcare provider. Put the large needle cover on needle. Unscrew the needle all the way by giving the needle cover 3 to 5 counter-clockwise turns. Pull off the covered needle and throw away as instructed by your healthcare provider. Push the blue cap back on, and put your TEVA-TERIPARATIDE INJECTION PEN in the refrigerator. <p>You can prevent this problem by always using a NEW needle for each injection, and by pushing the black injection button all the way in and slowly counting to five.</p>
<p>B. How can I tell if TEVA-TERIPARATIDE INJECTION PEN works?</p>	<p>→ The TEVA-TERIPARATIDE INJECTION PEN is designed to inject the full dose every time it is used according to the directions in the <i>User Manual</i>. The black injection button should be all the way in to show that the full dose of medicine has been injected from the TEVA-TERIPARATIDE INJECTION PEN.</p> <p>Remember to use a new needle every time you inject to be sure your TEVA-TERIPARATIDE INJECTION PEN will work properly.</p>
<p>C. I see an air bubble in my TEVA-TERIPARATIDE INJECTION PEN.</p>	<p>→ A small air bubble will not affect your dose and it will not harm you. You can continue to take your dose as usual.</p>
<p>D. I cannot get the needle off.</p>	<p>→</p> <ol style="list-style-type: none"> Put the large needle cover on the needle. Use the large needle cover to unscrew the needle. Unscrew the needle all the way by giving the large needle cover 3 to 5 counter-clockwise turns. If you still cannot get the needle off, ask someone to help you.
<p>E. What should I do if I have difficulty pulling out the black injection button?</p>	<p>→ Change to a new TEVA-TERIPARATIDE INJECTION PEN to take your dose as instructed by your healthcare provider.</p> <p>This indicates that you have now used all the medication that can be injected accurately even though you may still see some medication left in the medicine cartridge.</p>

Cleaning and Storage

Cleaning Your [®]Teva-Teriparatide Injection Pen

- Wipe the outside of the [®]Teva-Teriparatide Injection Pen with a damp cloth.
- Do not place the [®]Teva-Teriparatide Injection Pen in water, or wash or clean it with any liquid.

Storing Your [®]Teva-Teriparatide Injection Pen

- Refrigerate the [®]Teva-Teriparatide Injection Pen immediately after use. Read and follow the instructions in the Consumer Information Insert on how to store the medication.
- Do not store the [®]Teva-Teriparatide Injection Pen with a needle attached as this may cause air bubbles to form in the medicine cartridge.
- Store the [®]Teva-Teriparatide Injection Pen with the blue cap on.
- Never store the [®]Teva-Teriparatide Injection Pen in the freezer.
- If the [®]Teva-Teriparatide Injection Pen has been frozen, throw the device away and use a new [®]Teva-Teriparatide Injection Pen.
- If the [®]Teva-Teriparatide Injection Pen has been left out of the refrigerator, do not throw the pen away. Place the pen back in the refrigerator and call Teva Canada Limited (see *Contact Information*).

Other Important Notes

- The [®]Teva-Teriparatide Injection Pen contains 28 days of medicine.
- Do not transfer the medication to a syringe.
- Read and follow the instructions in the Consumer Information insert for using the medicine.
- Check the [®]Teva-Teriparatide Injection Pen label to make sure you have the correct medicine and that it has not expired.
- Contact Teva (see *Contact Information*) if you notice any of the following:
 - The [®]Teva-Teriparatide Injection Pen appears damaged.
 - The medicine is not clear and colourless, or if it has particles, do not use it.
- Use a new needle for each injection.
- During injection, you may hear one or more clicks – this is normal device operation.
- Do not share your medication.
- The [®]Teva-Teriparatide Injection Pen is not recommended for use by the blind or visually impaired persons without the assistance of a person trained in the proper use of the device.
- Keep your [®]Teva-Teriparatide Injection Pen out of the reach of children.

Disposal of Pen Needles and Delivery Device

Disposal of Pen Needles and the [®]Teva-Teriparatide Injection Pen Device

- Before disposing of the [®]Teva-Teriparatide Injection Pen device, be sure to remove the pen needle.
- Dispose of your [®]Teva-Teriparatide Injection Pen and used needles as directed by your healthcare professional and local institutional policies.
- Dispose of the device 28 days after first use.

Contact Information

If you have questions or need help with your [®]Teva-Teriparatide Injection Pen Device, contact the Teva-Teriparatide Patient Care program by:

Phone: 1.877.445.6984

Email: Teriparatidepatientcare@teva-canada.com